

# Homemade Cloth Face Masks

## Assembly Guide and Patterns

### Fabric

This mask has two layers—the inner layer touching the face and the outer layer. The two layers can be made of the same fabric or different fabrics. Use 100% cotton or a cotton blend for the best filtration and breathability.

It takes two pieces of fabric about 14.5 in. x 7.5 in. (36 cm. x 18 cm.) to make one adult mask (see Figure 1).

There are three sizes: Large, Medium, and Small. Patterns are available at the end of these instructions.

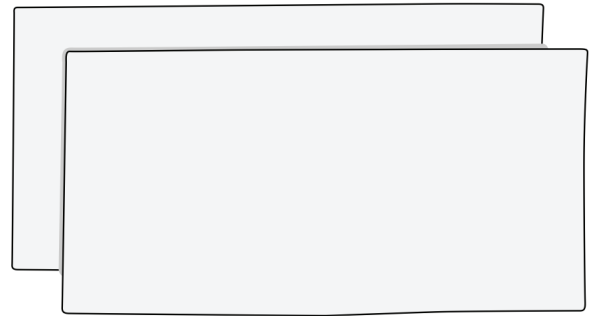


Figure 1

### Ear Loop Material

Each mask needs two strips for ear loops. Ear loop material should be less than ½ inch (1.25 cm) wide. Some options are elastic, elastic cord, string, ribbon, bias tape, or even shoelaces.

If using elastic, each piece should be 12 inches (35.5 cm.) long. For other materials each piece should be 15 inches (38 cm) long (see Figure 2).

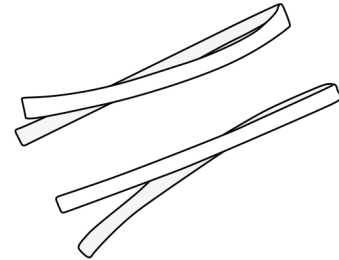


Figure 2

### Sewing Equipment

- Sewing machine
- Scissors or rotary cutter
- Thread
- Straight pins
- Safety pins

### Work Area

Before you start sewing, ensure you are symptom free. Wash and sanitize your hands and your work area thoroughly with a disinfectant (it must indicate that it kills viruses) per the instructions on the label. Ensure there are no potential contaminants (e.g., pet hair, food, etc.) in the work area.

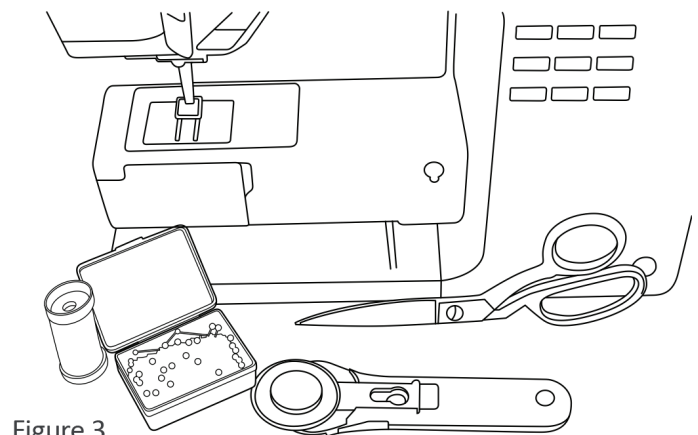


Figure 3



**ProjectProtect**

*In partnership with Intermountain Healthcare,  
University of Utah Health and Latter-day Saint Charities*

For more information, visit

**chlb.me**

## Step 1

### Cut out the two main mask pieces.

- Fold the fabric in half. On the folded edge of the fabric, place the pattern edge marked “place on fold” (see Figure 4).
- Pin the pattern on the fold of the fabric and cut it out.
- Repeat step 1 to cut the second piece.

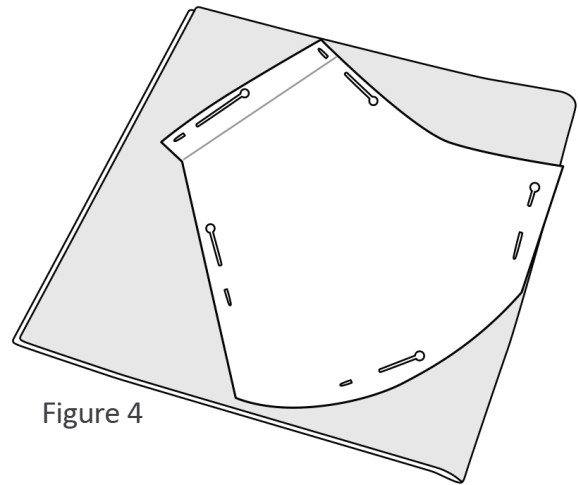


Figure 4

## Step 2

### Make the chin part of the mask.

- Open one of the cutout mask pieces. Fold it in half on the fold line, with the right sides inside.
- Sew along the chin seam, stitching 1/4 in (0.7 cm) from the edge (see Figure 5).
- If desired, press the seam to the side to help it lay flat.
- Fold and sew the second mask piece in the same way.

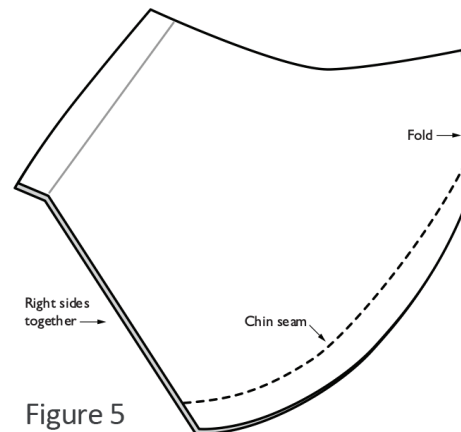


Figure 5

## Step 3

### Join the inner and outer layers together.

- Lay the two mask pieces on top of each other, with right sides of the fabric together (see Figure 6).
- Sew the two layers together across the top of the mask, using a 1/4 in (0.7 cm) seam.
- Sew the two layers together across the bottom of the mask.

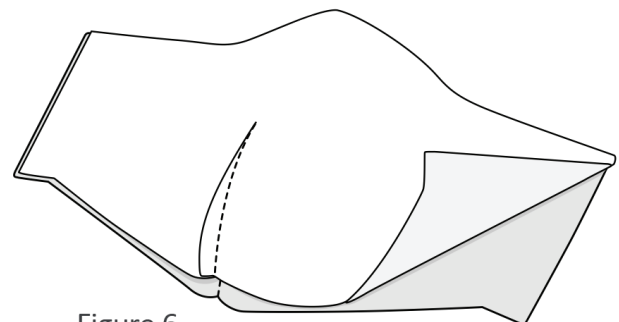


Figure 6

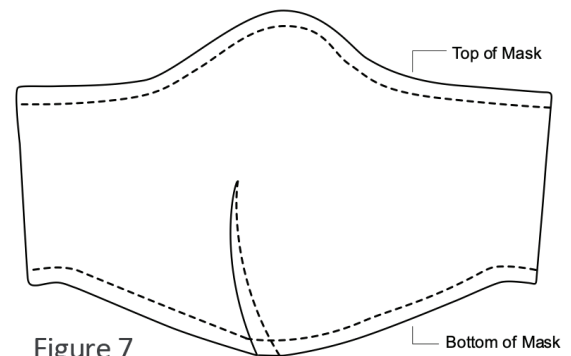


Figure 7



**ProjectProtect**

*In partnership with Intermountain Healthcare,  
University of Utah Health and Latter-day Saint Charities*

For more information, visit

**chlb.me**

## Step 4

### Turn the mask right-side out.

- Press with an iron.

## Step 5

### Topstitch along the top and bottom edges.

- Keep your stitching close to the edges.
- Do not stitch across the sides of the mask.

## Step 6

### Make the casing (a hollow channel) for the ear loop to go through.

On each side:

- Fold the raw side edges toward the inside of the mask about  $\frac{1}{4}$  in (0.7 cm) (See Figure 10).
- Then fold over again, about  $\frac{5}{8}$  in (1.5 cm).
- Pin in place and stitch as shown in Figure 11.

## Step 7

### Thread the ear loop through the casing along the sides of the mask.

Use a large-eye plastic needle or attach the end of the elastic to a small safety pin. Push the elastic through the casing on both sides of the mask (see Figure 12).

## Step 8

### Make loops the right size.

Tie the loop ends loosely. The wearer can adjust the length by tying a knot (see Figure 13).

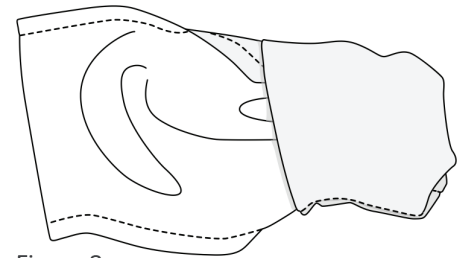


Figure 8

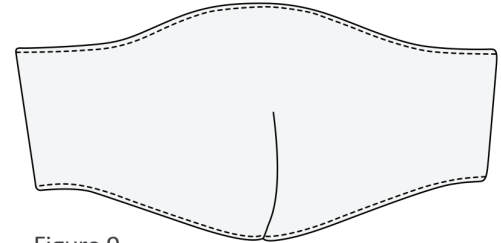


Figure 9

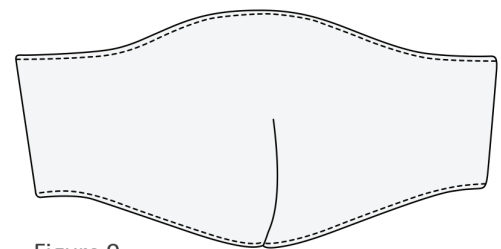


Figure 9

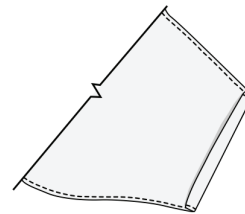


Figure 10

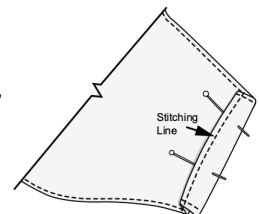


Figure 11

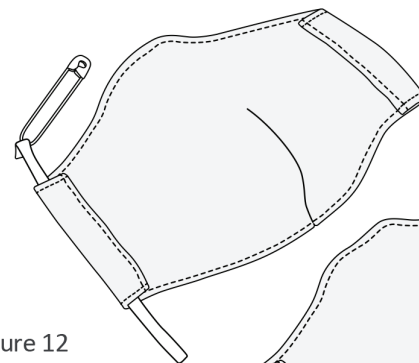


Figure 12

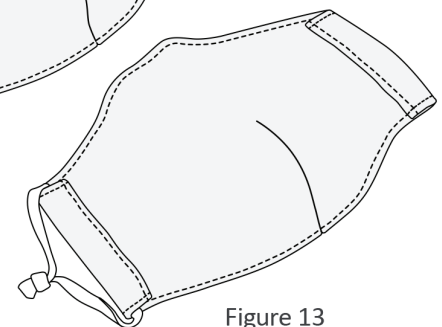


Figure 13



*ProjectProtect*

*In partnership with Intermountain Healthcare,  
University of Utah Health and Latter-day Saint Charities*

For more information, visit  
**chlb.me**

Sew with 1/4 in (6 mm) seam allowance.

Sew with 1/4 in (6 mm) seam allowance.




*Fabric Quantity:* 14.5 in x 7.5 in  
(36 cm x 18 cm).

*Notions:* Elastic, elastic cord, string, or ribbon [Loop material must be less than 1/2 in (1.25 cm) wide.]

2 in or 5 cm

# Medium Face Mask

Sew with 1/4 in (6 mm) seam allowance

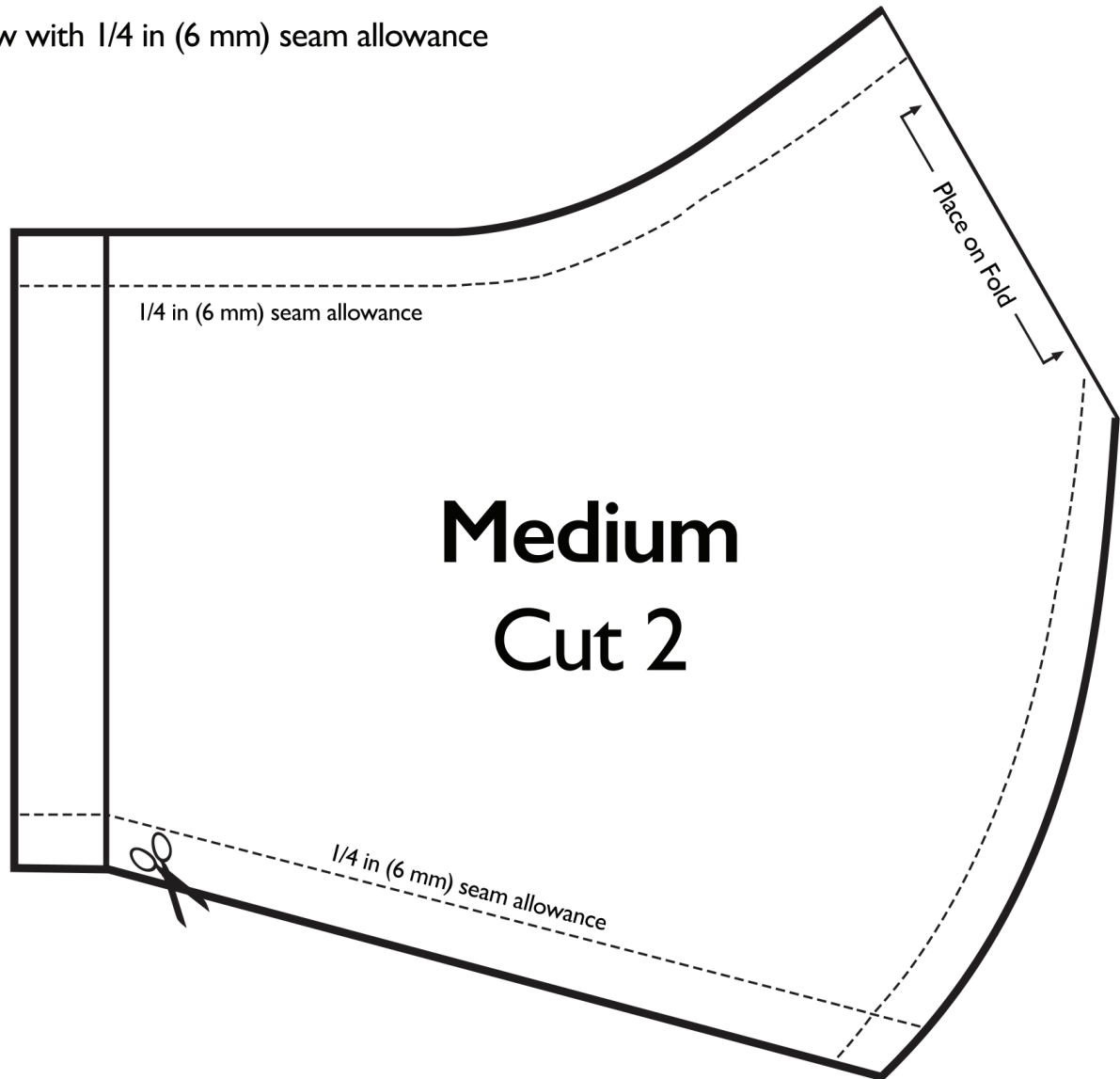


## Fabric and Notions Needed.

*Fabric Quantity:* 14.5 in x 7.5 in  
(36 cm x 18 cm).

The best fabric is cotton T-shirts, pillow cases, or other cotton material.


*Notions:* Elastic, elastic cord, string, or ribbon [Loop material must be less than 1/2 in (1.25 cm) wide.!

When printed, this line should be 2 in (5 cm). Enlarge or reduce as necessary to ensure your pattern is accurate.

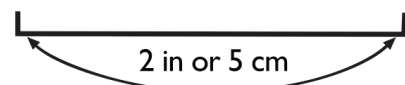

# Small Face Mask

Sew with 1/4 in (6 mm) seam allowance

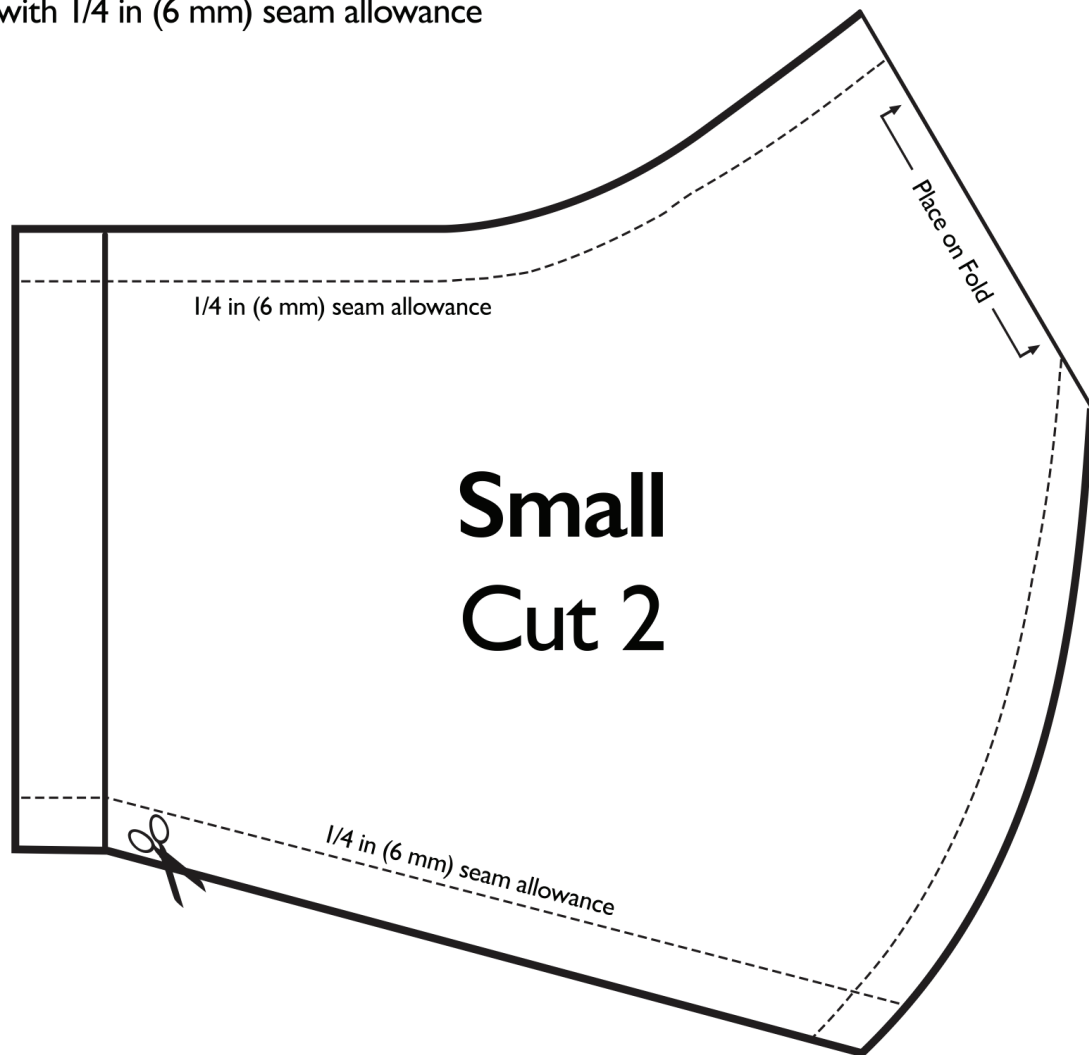

## Fabric and Notions Needed.

*Fabric Quantity:* 14.5 in x 7.5 in  
(36 cm x 18 cm).

The best fabric is cotton T-shirts, pillow cases, or other cotton material.

*Notions:* Elastic, elastic cord, string, or ribbon [Loop material must be less than 1/2 in (1.25 cm) wide.]

When printed, this line should be 2 in (5 cm). Enlarge or reduce as necessary to ensure your pattern is accurate.

